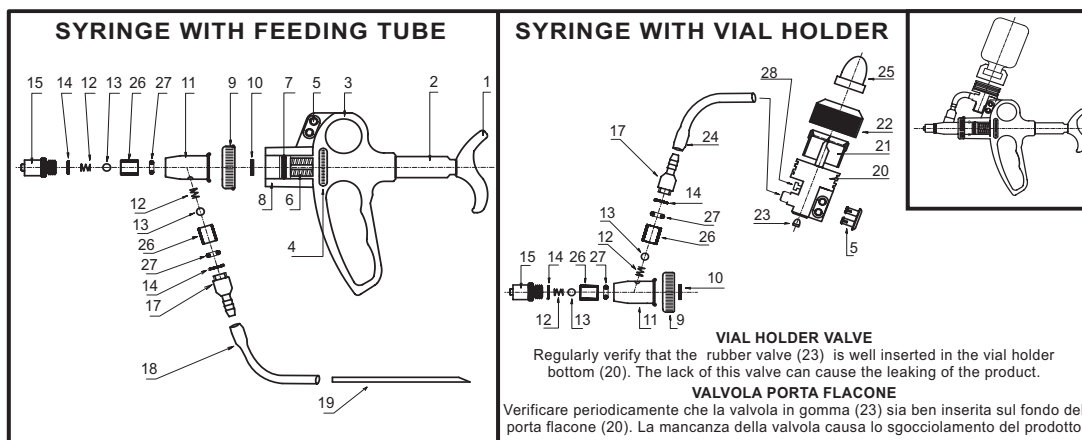
PARTS LIST

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SYRINGES RANGE

Art.0541 Capacity 0,5CC. - Adjustable dose from 0,1 CC TO 0,5CC.

- 0541.1 SYRINGE + TUBING SET
- 0541.2 SYRINGE + BOTTLE ATTACHMENT
- 0541.3 SYRINGE + TUBING SET + BOTTLE ATTACHMENT

Art. 0542 Capacity 1CC. - Adjustable dose from 0,2CC TO 1CC.

- 0542.1 SYRINGE + TUBING SET
- 0542.2 SYRINGE + BOTTLE ATTACHMENT
- 0542.3 SYRINGE + TUBING SET + BOTTLE ATTACHMENT

Art. 0543 Capacity 2CC. - Adjustable dose from 0,5CC TO 2CC.

- 0543.1 SYRINGE + TUBING SET

0543.2 SYRINGE + BOTTLE ATTACHMENT

0543.3 SYRINGE + TUBING SET + BOTTLE ATTACHMENT

Art. 0544 Capacity 5cc. - Adjustable dose from 0,5CC TO 5CC.

- 0544.1 SYRINGE + TUBING SET
- 0544.2 SYRINGE + BOTTLE ATTACHMENT
- 0544.3 SYRINGE + TUBING SET + BOTTLE ATTACHMENT

Art. 0545 Capacity 10cc. - Adjustable dose from 1CC TO 10CC.

- 0545.1 SYRINGE + TUBING SET
- 0545.2 SYRINGE + BOTTLE ATTACHMENT
- 0545.3 SYRINGE + TUBING SET + BOTTLE ATTACHMENT

INSTRUCTIONS FOR TUBING SET

Attach one end of the tubing (18) to the tubing connector (17) and the other end of the tubing to the feeding needle (19). Insert the feeding needle (19) into the bottle. In order to get out the air, it is recommended to pierce with a needle the cap next to the tube. Attach the injection needle (not supplied) to the Luer Lock (15). Fill in the syringe by repeatedly pushing the handle (1) until the liquid fills the barrel. Turn the dosage wheel (4) until the piston (6) lines up with the required graduation. To remove any remaining air bubbles hold the syringe with needle pointing up and push the handle slightly a few times.

INSTRUCTIONS FOR BOTTLE HOLDER

Unscrew the bottle lock (22) without removing it from the holder (20). Remove the protection cap (25). Insert into the holder (20) the medicine bottle by pressure and lock it by screwing the bottle lock (22). In order to avoid product leaking it is suggested to keep the bottle in the upward position and insert the holder with the syringe up side down. Fill the barrel by pushing the handle (1). Turn the dosage wheel (4) until the piston lines up with the required graduation. Attach the needle (not supplied) to the Luer Lock (15). To remove any remaining air bubbles, hold the syringe with needle pointing up and push the handle slightly a few times. NOTE: It is not necessary to dismount the bottle holder if the syringe is used with the tubing set, in this case it is important that the holder is protected by the cap (25).

TO SWITCH FROM FEED TUBE TO BOTTLE HOLDER

Unscrew slightly the valve locking (9) and turn 180° the valve, therefore tighten again the valve locking. Remove the feeding tube (18). Fix the Bottle Holder on the syringe handle locking it by the hook (5). Connect the tube (24) to the tube connector (17). Follow the above instructions for bottle holder.

MAINTENANCE

After each use, fill the syringe with water in order to remove any product residue in the valve. Periodically disassemble the springs and balls inside the valve, being careful not to loose them and to assemble them in the correct position (see scheme). After cleaning and drying the syringe, a few drops of oil must be applied into the barrel, on the valve balls and around the 2 piston o'rings. When reassembling is better to respect the position of the o'rings, black on the front and red on the back of the piston.

WARNING

In order to avoid product leaking it is suggested, in case of the vial holder syringe insert the holder with the syringe upside down and in case of feeding tube insert the needle, always keeping the bottle in the upward position. The screw air valve (28) is already calibrated. Adjust it only to increase or reduce the air inlet inside the bottle.

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